

Dark Stain Removal Kit

DESCRIPTION OF IWHITE DARK STAINS:

White Dark Stains is an easy-to-use teeth whitening product that is safe for your teeth and specifically formulated with Dark Pigment Adsorption Technology.

This at-home treatment should be worn for only 20 minutes a day for instant results, leaving your teeth looking up to 8 shades whiter. The product can be used for 5 consecutive days for even more astonishing results or you can use it just once in a while whenever you feel it is necessary. The one-size-fits-all trays are prefilled with active gel and hygienically packed. One packet contains 10 prefilled trays which are ready to use. Wear one tray on your upper dental arch and one on your lower dental arch.

- It **ACTIVELY REMOVES COFFEE, TEA, TOBACCO & WINE STAINS** from the surface of the teeth without harming the enamel: Thanks to the Dark Pigment Adsorption technology, this innovative whitening system provides an immediate whitening effect.
- It helps to **REDUCE BAD BREATH** by ADSORBING ODOURS left by COFFEE, TEA, TOBACCO & WINE.
- Its **ANTI-STAIN** formula effectively helps prevent the formation of new stains by impeding the adhesion of the dark pigments from COFFEE, TEA, TOBACCO & WINE onto the tooth surface.
- It provides a highly available source of calcium and therefore **STRENGTHENS** the teeth.
- It's a **ONE-STEP SYSTEM**: no need to clean or boil mouthpieces, mess with syringes or brushes and measure gel.
- The tray is designed to **WHITEN BOTH THE FRONT AND THE BACK OF THE TEETH**. This is important because teeth are translucent and stains on the back can show through the front.
- It's **EASY TO WEAR AND TO USE**: the one-size- fits-all trays are soft and flexible.
- There is **NO MIXING** involved, each tray is prefilled with just the right amount of whitening gel, which is formulated specifically to prevent sensitivity.
- The iWhite technology is clinically proven to be **SAFE** and **EFFECTIVE**.

INDICATIONS:

iWhite Dark Stains was designed to be used in case of dental discolorations due to smoking, drinking tea, coffee and/or red wine.

INSTRUCTIONS OF USE:

1. Brush teeth gently before each application.
2. Make sure that your hands are clean and dry.
3. Remove the prefilled tray from the sealed sachet and immediately place the tray in your mouth ensuring the gel is in contact with the tooth surface. Activate the gel by pushing the tray towards the teeth. Upper and lower teeth can be whitened at the same time.
4. Gently close your lips over the tray and carefully close your mouth. Caution: biting too hard may cause excess gel to ooze out of the tray and away from the teeth.
5. Remove the tray after 20 minutes and discard immediately. Thoroughly rinse the remaining gel from your teeth and gums with water. If any of the microparticles are left behind between your teeth, you can easily remove them with dental floss.



PRECAUTIONS:

- Keep out of the reach of children.
- Not recommended for pregnant or breast-feeding women.
- Not recommended for children under 14 years.
- Avoid swallowing the gel.
- Whitens natural teeth only. Fillings, crowns and veneers will not be affected by this product.
- Do not use the product when wearing braces, retainers, tooth jewelry and/or piercings.
- Do not use this product if you suffer from gum problems. Discontinue the treatment if you experience pain or excessive sensitivity; if necessary contact your pharmacist.
- Do not use this product within 2 weeks of dental and/or orthodontic treatments (fillings, crowns, implants and so on).
- Do not lie down while carrying out the treatment.
- Use of the product is inadvisable if you are allergic to one of the ingredients.
- Not recommended for extended contact with human skin.
- Rinse thoroughly with water if the product comes into contact with the eyes.
- Avoid getting the product on clothing.
- The prefilled trays are for single use only. If the packaging of a tray has been damaged, it should not be used.
- Once an individual tray has been opened, the product must be used within 30 minutes.
- For optimal results, avoid the consumption of acidic food and beverages such as lemonade, cola etc. as well as tobacco, coffee, red wine, fruit juice and other staining substances during the treatment period.
- We recommend visiting your dentist every 6 months for good oral hygiene.
- Store in a cool dry place between 2°C and 25°C away from heat and direct sunlight.
- Do not use after the expiry date on the packaging.

OTHER INFORMATION:

Manufacturer:
Sylphar nv
Xavier De Cocklaan 42
B-9831 Deurle
Belgium
Tel +32 9 321 70 80
Fax +32 9 321 70 90
www.iwhiteinstant.com

Ingredients: Aqua, Hydrated Silica, Glycerin, Sodium Hexametaphosphate, Sorbitol, Mannitol, Chondrus Crispus Powder, Charcoal Powder, PEG-40 Hydrogenated Castor Oil, Aroma, Phthalimidoperoxycaproic Acid, Microcrystalline Cellulose, Zea Mays Starch, Sucrose, Hydroxypropyl Starch Phosphate, Diazolidinyl Urea, Methyl Diisopropyl Propionamide, Acrylates/Acrylamide Copolymer, Paraffinum Liquidum, Xylitol, Monosodium Citrate, Xanthan Gum, PVM/MA Copolymer, Calcium Gluconate, Calcium Lactate, Hydrogenated Lecithin, Potassium Acesulfame, Polysorbate 85, C12-15 Pareth-3, C177007, C177891.

Content:
10 x 0,8 gr